

# UltraClean II Bench Top Washer Disinfector Dryer





## UltraClean II

### Bench Top Washer Disinfector Dryer

Designed to protect dental handpieces:



The special detergent is formulated to achieve excellent cleaning results and has a neutral pH level. This, when combined with an optimum temperature of 90°C during the thermal disinfect cycle, means that the metals and coatings on your dental instruments will be protected for longer.

Forced air drying ensures that the inside lumens (hollows) of the handpieces are as dry as the outside. This means that the subsequent oiling of the instrument is not compromised – washer dryers without forced air may leave behind water droplets which will repel the oil coating.

#### More Features and benefits...

- Fully compliant to EN 15883 and HTM2030 – the recommended ‘best practice’ standard – to give you real peace of mind
- Seamless hands free process from the dentist, through cleaning to sterilization in the autoclave, eliminating sharps injuries and instrument damage
- Full load wash, thermal disinfect and drying cycle in less than 60 minutes for fast and effective throughput of instruments
- High load capacity - the equivalent of 6 dental trays per cycle – for cost effective cleaning
- A choice of 3 cycles – small load, normal load and a double wash for heavily contaminated instruments – all with a guaranteed compliant performance for high precision and maximum energy efficiency
- Forced hot air drying removes all moisture on instruments making them sterilization-ready for the autoclave
- Self-dosing detergent delivers consistent results - reduces waste and costs less per cycle
- Unique air gap prevents backflow pollution to ensure the safe disposal of contaminated fluid as required by BS EN 1717:2001\*
- Can be supplied with optional data logger or printer for easy, recordable data capture – essential for compliance

\* Protection against pollution of potable water in water installations and general requirements of devices to prevent pollution by backflow

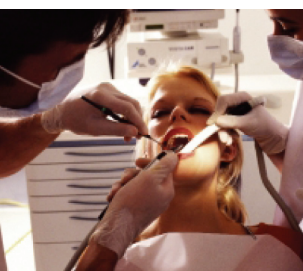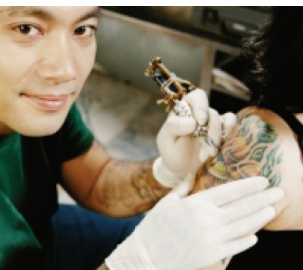

# UltraClean II



## Technical Specification

Product Name: UltraClean II Bench Top

Product Code: 800023

### Dimensions

Width	560 mm
Height	555 mm
Depth	575 mm
Max net weight	43 kg
Gross weight	46 kg
Chamber width	400 mm
Chamber depth	450 mm
Chamber height	300 mm

### Load capacity

Chamber volume	60 litre
Basket volume	50 litre

Process Monitoring	✓
Fault alert	✓

### Warranty

Parts (Unlimited usage, excludes consumables)	1 Year
---	--------

### Connections


Drain connection	25 mm dia
Voltage	230 VAC
Wattage	2750 W
Frequency (hz)	50 Hz
Current	12A
Power pump	150 W

hydraulic connection cold water	3/4" male
min / max pressure	2.5 Bar min 5 Bar max
min / max temperature	+5 to +15 degrees C

hydraulic connection warm water	3/4" male
min / max pressure	2.5 Bar min 5 Bar max
maximum capacity	8 KG Load
min / max temperature	+5 to +15 degrees C

Noise	52 dB(A)
Max room temperature admitted	40 degrees C

## Choice of 3 compliant wash programmes

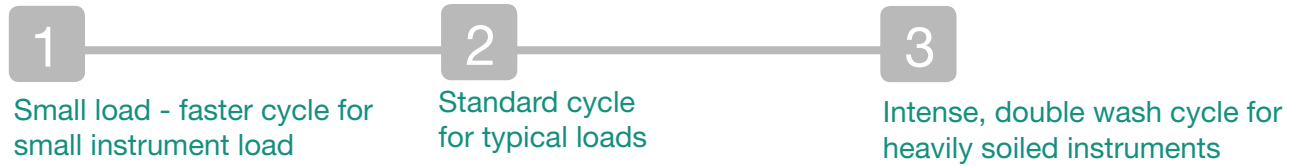


## Stages within a wash cycle


1: Short Cycle



2: Standard Cycle

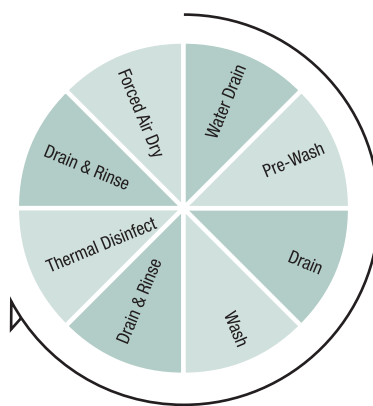


3: intensive Cycle

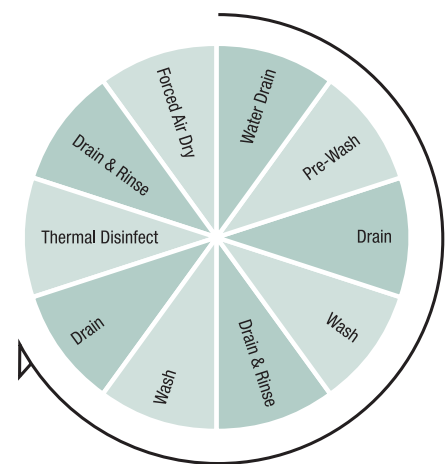


## Furniture included


8-position tray set

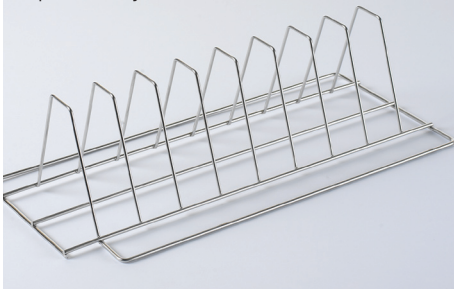

Net basket for small items

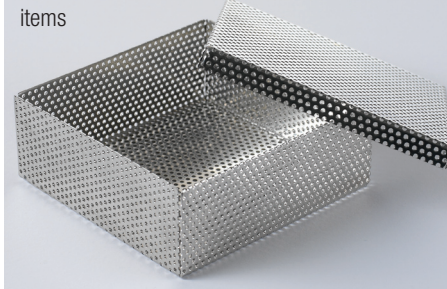

Vertical basket for mirrors and instruments

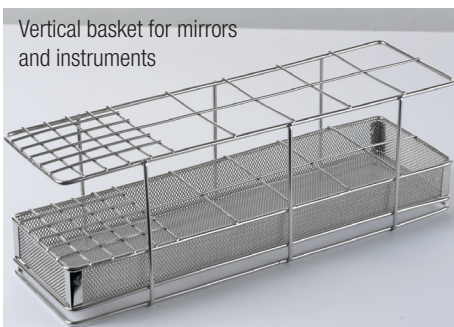

Basket with 8 hand-piece connections



# Infection Control?

## 3 easy steps to better practice with Prestige Medical

The UltraClean II Washer Disinfector Dryer is just one key step in your quest to comply with and exceed the guidelines for infection control.

Prestige Medical provides **Integrated decontamination solutions** - products and services to help you at every step of your infection control procedure of washing, sterilizing and recording data.



## Sterilization as easy as 1-2-3

### Enjoy the benefits of a fully integrated decontamination solution:

Prestige Medical products all work beautifully together and are designed to give you a smooth, hands-free sterilization process. This ensures the highest standards of sterilization and infection control, giving you peace of mind in your day-to-day business operation.



Use a fully compliant washer disinfector dryer such as the UltraClean II



Sterilize effectively with the correct vacuum or non-vacuum autoclave from Prestige Medical



Record the cycle data following the correct procedures as specified in HTM 01-05



= full compliance

Compliant to International standards and Guidelines BS EN 13060:2004 and EN 15883

Contact us for a free, independent infection control risk assessment of your premises, or for advice on decontamination rooms.

## Prestige Medical - integrated decontamination solutions

For over 20 years, Prestige Medical has been respected as a class leader in the development and manufacture of sterilization and disinfection equipment. With over 1 million units delivered into over 80 countries worldwide, we have built an enviable reputation for innovation, performance and trust.

With infection control standards becoming increasingly stringent, a new breed of equipment is now required that is capable of meeting these challenges. The UltraClean II Washer Disinfector helps you surpass these standards. As part of our new generation of Integrated Decontamination Solutions we are able to take infection control to even greater levels. Our 3 step system has products and services that give you confidence at every stage of your decontamination and sterilization procedures.

Additionally, all Prestige Medical products and services are backed by a team of expert technicians, service engineers and customer support staff. Their mission is to deliver the most effective solution, impart their specialist knowledge of dental infection control and design a system that's exactly right for you.

### standards and guidelines

Designed and manufactured to comply with the following standards and guidelines:

- 93/42/EEC Medical Device Directive
- EN15883-1
- HTM 2030



Prestige Medical Limited  
East House, Duttons Way,  
Shadsworth Business Park,  
Blackburn BB1 2QR

Tel : +44 (0) 1254 682 622  
Fax : +44 (0) 1254 682 606

[www.prestigemedical.co.uk](http://www.prestigemedical.co.uk)  
[sales@prestigemedical.co.uk](mailto:sales@prestigemedical.co.uk)

Registered in England  
Reg No. 2826793  
Vat No. 633707835

Registered Office:  
Unit 1, First Avenue  
Maybrook Industrial Estate  
Minworth, Sutton Coldfield  
West Midlands, B76 1PA

Prestige Medical is a registered trademark



contact us for an  
infection control  
risk assessment  
questionnaire